CHU | UVC
BRUGMANN

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CHU | UVC
BRUGMANN

[Informatie](#)

CHU|UVC BRUGMANN

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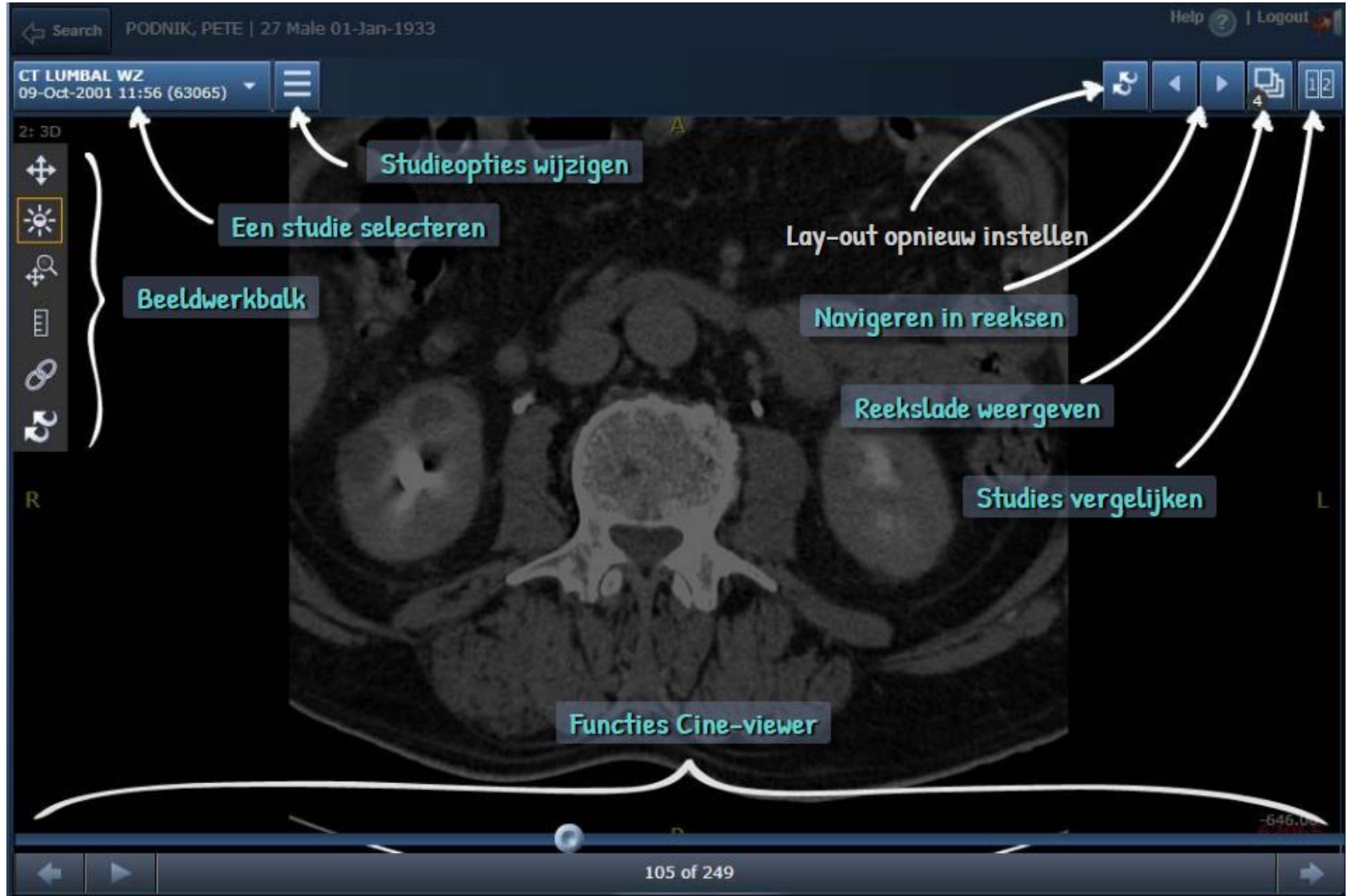
HOOFDMENU

WEERGAVE OPTIES

The screenshot displays a medical imaging software interface with a dark theme. At the top, a search bar shows 'PODNIK, PETE | 27 Male 01-Jan-1933'. Below it, the study name 'CT LUMBAL WZ' and date '09-Oct-2001 11:56 (63065)' are visible. A central menu is open, listing several options:

- REPORTS**
 - Study Comments (checked)
- LAYOUT**
 - 1x1 Layout (selected)
 - 2x1 Layout
 - 2x2 Layout
- SHARE STUDY**
 - Share Study (QR code icon)
- IMPORTANT IMAGES**
 - For Patient (checked)

Annotations in Dutch point to these options:

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Other interface elements include a 'Startpositie' button at the bottom right, a '105 of 249' page indicator at the bottom center, and navigation controls at the bottom left.

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R

A

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The screenshot displays a medical imaging software interface. At the top, a search bar shows 'PODNIK, PETE | 27 Male 01-Jan-1933'. Below it, the study name 'CT LUMBAL WZ' and date '09-Oct-2001 11:56 (63065)' are visible. The main window shows a 2D axial CT scan of the lumbar spine. A blue box labeled 'Startpositie' is positioned over the vertebral body. A blue slider at the bottom indicates the current image number '105 of 249'. Navigation controls include arrows for 'Eén beeld achteruit gaan' (one image back), 'Afspelen' (play), 'Verslepen om te navigeren door de reeks' (drag to navigate), and 'Naar volgende beeld gaan' (to next image). The interface also shows 'R' and 'L' markers for right and left sides, and a 'Help | Logout' button in the top right corner.

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-646,00

NAVIGATIE IN CINEVIEW MODUS

The screenshot displays the Cineview interface for a CT scan. At the top, the search bar shows 'PODNIK, PETE | 27 Male 01-Jan-1933'. Below it, the scan details are 'CT LUMBAL WZ' and '09-Oct-2001 11:56 (63065)'. The main window shows a 3D axial CT scan of the lumbar spine. A blue box labeled 'Startpositie' is centered on the vertebra. The interface includes a left sidebar with navigation icons (home, zoom, pan, scroll, rotate) and a bottom control bar. The control bar features buttons for 'Heen-en-weer' (Home) and 'Herhalen' (Repeat), a 'Pauze' (Pause) button, a timeline with 'Bijknippen links' (Trim left) and 'Bijknippen rechts' (Trim right) markers, and a 'Waarschuwing: trage beeldsnelheid' (Warning: slow image speed) indicator. A vertical slider on the right is labeled 'Afspeelsnelheid aanpassen' (Adjust playback speed) and is set to 30. The bottom status bar shows '105 of 249'.

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105 of 249

30

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The screenshot displays a medical imaging software interface for a CT scan. The main window shows a cross-sectional view of the lumbar spine. On the left side, there is a vertical toolbar with several icons. White arrows point from these icons to text boxes containing Dutch instructions. The interface also includes a search bar at the top left, patient information, a help and login section at the top right, and a status bar at the bottom.

Search: PODNIK, PETE | 27 Male 01-Jan-1933

CT LUMBAL WZ
09-Oct-2001 11:56 (63065)

2: 3D

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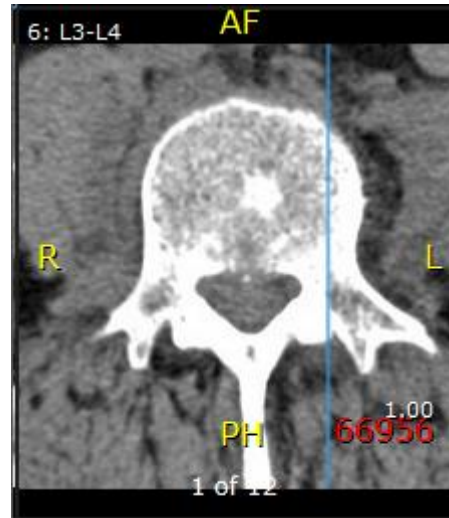
Helderheid/contrast (Window/Level) instellen

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Copyright (c) 2015 Agfa HealthCare N.V., Septestraat 27, B-2940, Mortsel, Belgium.