CHU | UVC
BRUGMANN

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CHU | UVC
BRUGMANN

[Informatie](#)

CHU|UVC BRUGMANN

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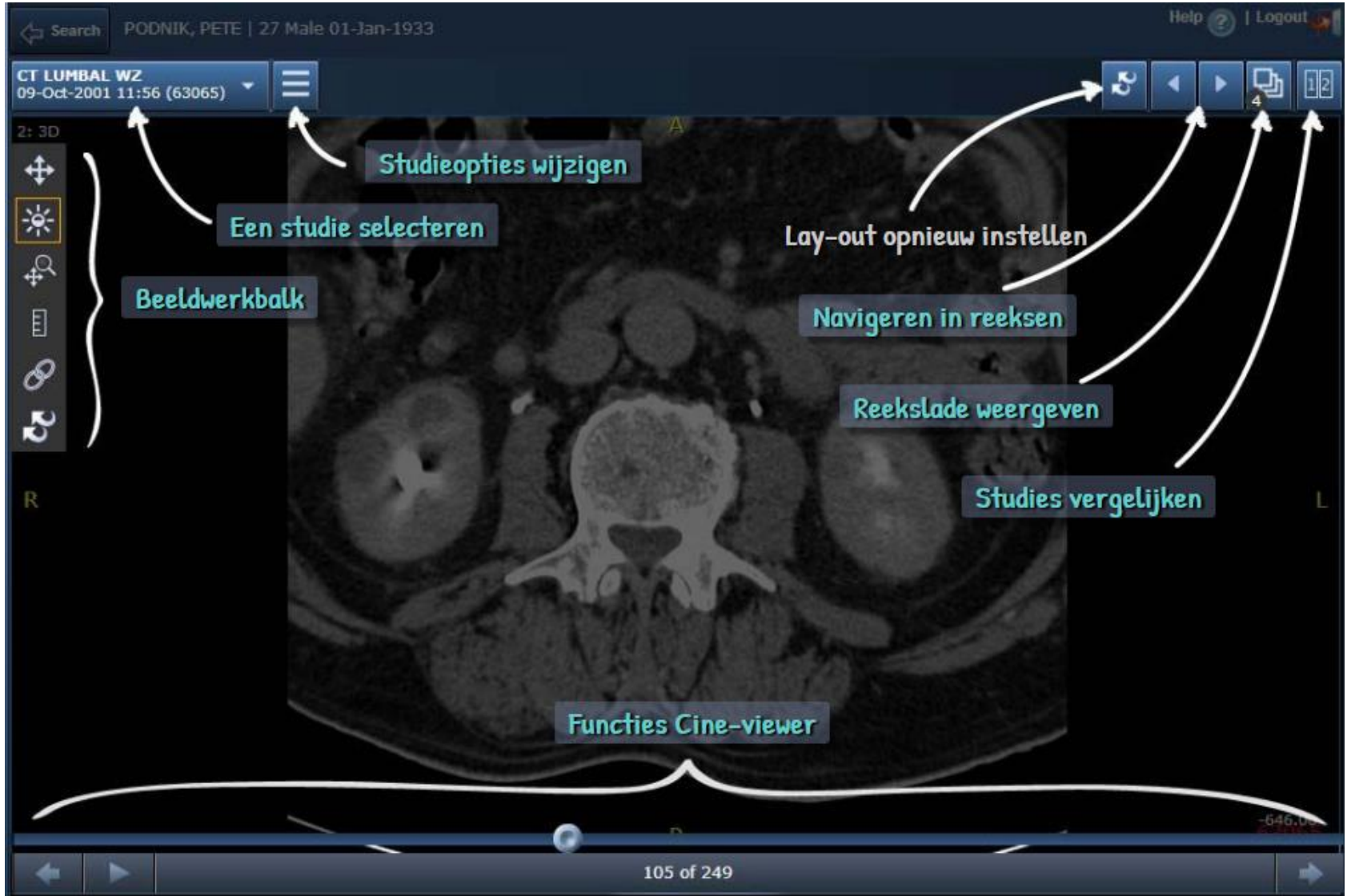
HOOFDMENU

WEERGAVE OPTIES

The screenshot displays a medical imaging software interface with a dark theme. At the top, a search bar shows 'PODNIK, PETE | 27 Male 01-Jan-1933'. Below it, the study name 'CT LUMBAL WZ' and date '09-Oct-2001 11:56 (63065)' are visible. A central menu is open, listing several options:

- REPORTS**
 - Study Comments (checked)
- LAYOUT**
 - 1x1 Layout (selected)
 - 2x1 Layout
 - 2x2 Layout
- SHARE STUDY**
 - Share Study (QR code icon)
- IMPORTANT IMAGES**
 - For Patient (checked)

Annotations in Dutch point to these options:

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Other interface elements include a 'Startpositie' button at the bottom right, a '105 of 249' page indicator at the bottom center, and navigation controls at the bottom left.

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-646,00

R

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The screenshot displays a medical imaging software interface. At the top, a search bar shows 'PODNIK, PETE | 27 Male 01-Jan-1933'. Below it, the study name 'CT LUMBAL WZ' and date '09-Oct-2001 11:56 (63065)' are visible. The main window shows a 2D axial CT scan of the lumbar spine. A blue box labeled 'Startpositie' is positioned over the vertebral body. A blue slider at the bottom indicates the current image number '105 of 249'. Navigation controls include arrows for 'Eén beeld achteruit gaan' (one image back), 'Afspelen' (play), 'Verslepen om te navigeren door de reeks' (drag to navigate), and 'Naar volgende beeld gaan' (to next image). The interface also shows 'R' and 'L' markers for right and left sides, and a 'Help | Logout' button in the top right corner.

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-646,00

NAVIGATIE IN CINEVIEW MODUS

The screenshot displays a medical software interface in Cineview mode. At the top, a search bar shows 'PODNIK, PETE | 27 Male 01-Jan-1933'. Below it, the scan details are 'CT LUMBAL WZ' and '09-Oct-2001 11:56 (63065)'. The main area shows a 3D axial CT scan of the lumbar spine. A blue box labeled 'Startpositie' is centered on the spine. The interface includes a left sidebar with navigation icons (home, brightness, zoom, list, scroll, refresh) and a right sidebar with playback controls (rewind, play, stop, next, list). At the bottom, a timeline shows the current frame '105 of 249'. A blue play button is active, and a yellow warning icon is visible. Annotations with arrows point to various controls: 'Heen-en-weer' and 'Herhalen' on the left; 'Pauze' on the play button; 'Bijknippen links' and 'Bijknippen rechts' on the timeline; 'Verslepen om te navigeren door de reeks' on the play button; 'Waarschuwing: trage beeldsnelheid' on the warning icon; and 'Afspeelsnelheid aanpassen' on the speed slider.

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105 of 249

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The screenshot displays a medical imaging software interface. At the top, a search bar shows 'PODNIK, PETE | 27 Male 01-Jan-1933'. Below it, the study information is 'CT LUMBAL WZ' with a date and time '09-Oct-2001 11:56 (63065)'. The main area shows a 2D axial CT scan of the lumbar spine. On the left side, there is a vertical toolbar with several icons. White arrows point from these icons to text boxes containing Dutch descriptions of the functions. The icons and their corresponding descriptions are: a four-way arrow for 'Beelden omdraaien of roteren', a sun icon for 'Helderheid/contrast (Window/Level) instellen', a magnifying glass with a plus sign for 'Zoomen of pannen', a ruler icon for 'Metingen schuifmaat', a circular arrow for 'Reeksen koppelen', and a square with a circular arrow for 'Alle transformaties resetten'. In the bottom right corner of the image area, there is a button labeled 'Startpositie' and a numerical value '-646.00'. At the very bottom of the interface, there is a navigation bar with left and right arrows and the text '105 of 249'.

Search PODNIK, PETE | 27 Male 01-Jan-1933 Help ? | Logout

CT LUMBAL WZ
09-Oct-2001 11:56 (63065)

2: 3D

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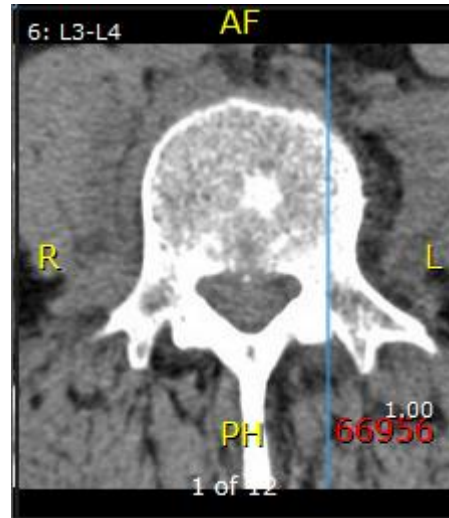
Helderheid/contrast (Window/Level) instellen

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Copyright (c) 2015 Agfa HealthCare N.V., Septestraat 27, B-2940, Mortsel, Belgium.